Rapid one-step screening test.

Integrated E-Z Split Key® Cup

Urine Drug Screening Device

For professional in vitro diagnostic use only

Knowing now matters.®

Dr. GERARDO ABIEL OVIEDO
Coord. Medicina Laboral
M.N. 99761 - M.N. T 5957
Linea Roca
Trenes Argentinos Operaciones

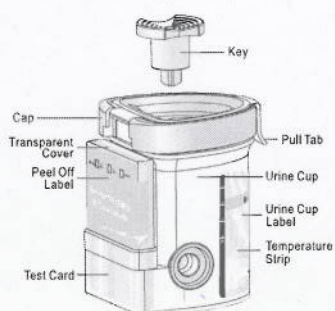
Lic. Hernán Bianchimano
Subgerente de RR.HH.
Linea ROCA
Operadora Ferroviaria S.E.

Integrated E-Z Split Key® Cup Urine Drug Screen

Features and Benefits

- Integrated test cup screens for up to 14 prescription and illicit drugs (3-12 panel options available)
- Begin testing at your convenience with unique key activator
- Fast test results in 5 minutes
- Self-contained collection cup and test help minimize exposure to urine
- Try the Integrated E-Z Split Key Cup A.D. for expanded specimen validity testing: CRE, GLUT, NIT, OX, pH, S.G.*


Test in 3 Easy Steps


1
Collect specimen and snap test lid shut.

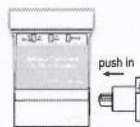2
Collector pushes the key to fully closed position to initiate the test.

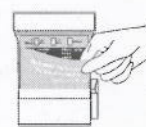3
Peel label to reveal test and specimen validity (if applicable). Read test results at 5 minutes.

5 min

PRODUCT PROCEDURE NOTE: Refer to product insert for complete instructions, limitations, and warnings. For healthcare professionals including professionals at point of care sites, immunoassay for in vitro diagnostic use only. This assay provides only a preliminary analytical test result. A more specific alternate chemical method must be used in order to obtain a confirmed analytical result. Gas chromatography-mass spectrometry (GC-MS) is the preferred confirmatory method. Clinical consideration and professional judgment should be applied to any drug of abuse test result, particularly when preliminary positive results are indicated.

Drug screen options and cutoff levels

Test	Calibrator	Cutoff Level
Amphetamine (AMP 1000)	d-Amphetamine	1,000 ng/mL
Amphetamine (AMP 300)	d-Amphetamine	300 ng/mL
Barbiturates (BAR)	Secobarbital	300 ng/mL
Benzodiazepines (BZO)	Oxazepam	300 ng/mL
Buprenorphine (BUP)	Buprenorphine	10 ng/mL
Cocaine (COC 300)	Benzoylcegonine	300 ng/mL
Cocaine (COC 150)	Benzoylcegonine	150 ng/mL
Marijuana (THC)	11-nor- Δ^9 -THC-9-COOH	50 ng/mL
Methadone (MTD)	Methadone	300 ng/mL
Methamphetamine (mAMP 1000)	d-Methamphetamine	1,000 ng/mL
Methamphetamine (mAMP 500)	d-Methamphetamine	500 ng/mL
Methylenedioxyamphetamine (MDMA - Ecstasy)	d,l-Methylenedioxyamphetamine	500 ng/mL
Opiate (MOP 300)	Morphine	300 ng/mL
Opiate (OPI 2000)	Morphine	2,000 ng/mL
Oxycodone (OXY)	Oxycodone	100 ng/mL
Phencyclidine (PCP)	Phencyclidine	25 ng/mL
Propoxyphene (PPX)	Propoxyphene	300 ng/mL
Tricyclic Antidepressants (TCA)	Nortriptyline	1,000 ng/mL

*Specimen Validity Tests: Creatinine (CRE), Glutaraldehyde (GLUT), Nitrite (NIT), Oxidants/PCC (OX), pH (pH), Specific Gravity (S.G.)

Alere Toxicology

Products 800.340.4029 | aleretoxicology.com

© 2016 Alere. All rights reserved. The Alere Logo, Alere, E-Z Split Key, and Knowing now matters are trademarks of the Alere group of companies. All other trademarks referenced are trademarks of their respective owners. MKT50276 REV1 10/16

Dr. GERARDO ARIEL OVIEDO
Coord. Medicina Laboral
M.N. 99761 - M.N. T 5957
Linea RoCA
Trenes Argentinos Operaciones

Lic. Hernán Branchimano
Subgerente de RR.HH.
Linea ROCA
Operadora Ferroviaria S.E.

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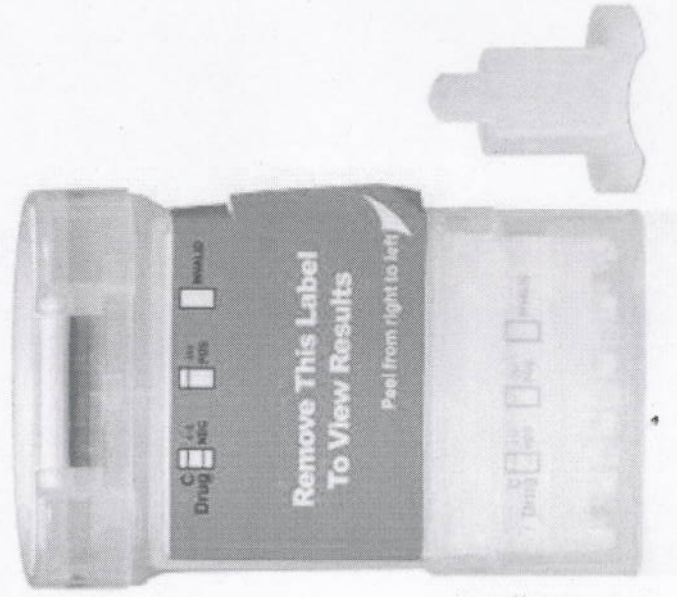
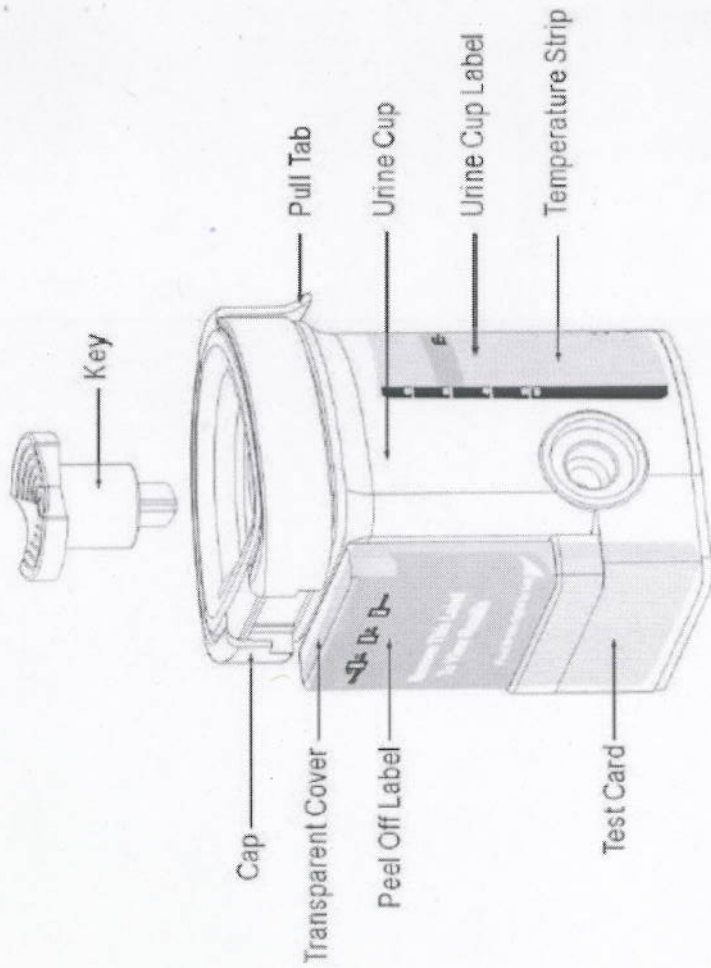
CONFIDENCIAL - FERRIVIA S.E.

Componentes

INTEGRATED E-Z SPLIT KEY® CUP II
 CATALOG NUMBER: DOA-XXXX-XX
 <COC>+<AMP>+<AMP>+<THC>+<CPT>
 LOT: DOAXXXXXXXXX EXP: 2010/09
 LXXXXXX-01

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Alere™

Niveles de Cut off

Drug	Target Drug	Cutoff
Amphetamine (AMP)	D-Amphetamine	1,000 ng/mL
Barbiturates (BAR)	Secobarbital	300 ng/mL
Benzodiazepines (BZO)	Oxazepam	300 ng/mL
Cocaine (COC)	Benzoyllecgonine	300 ng/mL
Marijuana (THC)	11-nor- Δ^9 -THC-9 COOH	50 ng/mL
Methadone (MTD)	Methadone	300 ng/mL
Methylenedioxy-methamphetamine (MDMA)	D,L Methylenedioxy-methamphetamine	500 ng/mL
Methamphetamine (M-AMP)	D-Methamphetamine	1,000 ng/mL
Opiates OPI 300 OPI 2000	Morphine Morphine	300 ng/mL 2,000 ng/mL
Oxycodone (OXY)	Oxycodone	100 ng/mL
Phencyclidine (PCP)	Phencyclidine	25 ng/mL
Propoxyphene (PPX)	Propoxyphene	300 ng/mL
Tricyclic Antidepressants (TCA)	Nortriptyline	1,000 ng/mL

Dr. GERARDO ARIEL OVIDIO
 Codic. Medicina Laboral
 M.N. 9976 / M.N. 7 5957
 Trenes Argentinos Operaciones

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